



## Lifecosc COVID-19 Antigen Test

Product Code	Product Name	Packing	Sample Type	Components
RC-HM01	Lifecosc™ COVID-19 Antigen Test Cassette	25T/Box	Nasopharyngeal swab; Anterior nasal swab; Oropharyngeal swab; Saliva	Test cassette*25 Buffer tube*25 Dropper*25 Flocked swab*25
RC-HM02	Lifecosc™ COVID-19 Antigen Test Cassette (Nasal Swab)	1T/Pack 24Packs/Box	Anterior nasal swab;	Test cassette*24 Buffer tube*24 Dropper*24 Flocked swab*24

## Lifecosc SARS-CoV-2 & Flu A/B Antigen Combo Test Cassette

Product Code	Product Name	Packing	Sample Type	Components
RC-HM04	Lifecosc™ Sars-CoV-2 & Flu A/B Antigen Combo Test Cassette	25T/Box	Nasopharyngeal swab; Oropharyngeal swab;	Test cassette*25 Buffer tube*25 Dropper*25 Flocked swab*25

COVID-19 and influenza are infectious disease caused by Sars-CoV-2 and Flu A/B respectively, however this 2 diseases got similar symptom while it could caused by infection of one of the virus, or co-infection simultaneously. To differentiate the cause of the symptom by a combo test will help medical service providers save diagnosis time and make accurate treatment suggestion.



### Operation Procedure


- Collect sample from nasopharyngeal and oropharyngeal swab according to instruction of use
- Insert the nasal swab into the tube which contains the extraction reagent. Rotate for at least 5 times
- Remove the swab while squeezing the sides of the tube against the swab, to release the liquid from the swab.
- Cover the tube with the provided lid tightly and insert the tube back into the box.
- Add 3 drops from the tube into each sample well by gently squeezing the sides of the tube.
- Read the test result at 15-20 minutes, DO NOT read the result after 20 minutes.

### Result Interpretation

#### Result of Sars-CoV-2 Antigen



#### Result of Influenza A/B Antigen

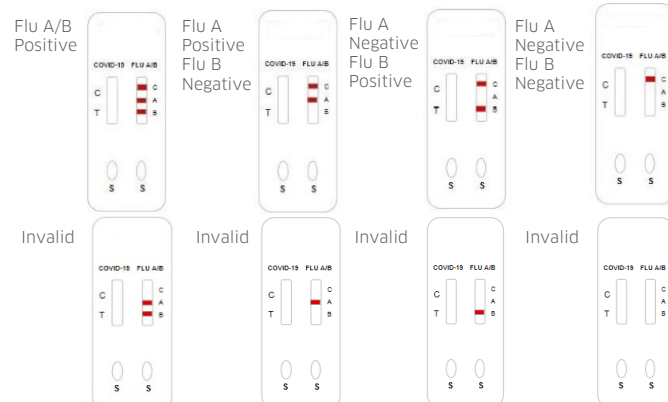

Lifecosc Biotech Limited

3B Fusen Building, 669 Shuangming Avenue, Guangming District, Shenzhen, China  
T: +86 755 2342 0845 E: info@lifecosc.com

